

Using healthPAL™

with the
ONETOUCH Ultra Glucose Meters



From the MedApps HealthPAL QuickStart Guide

Collecting Glucose Readings

1. After glucose readings have been collected, attach MedApps Smart Cable's 30-pin connector to HealthPAL and insert the alternate end of cable into the glucose meter's data port (**Note:** Glucose meter should NOT be connected to HealthPAL via Smart Cable while glucose measurements are collected). Ensure that the test strip has been removed, and the meter is OFF and not in "PC" mode prior to connecting to HealthPAL.
2. Once the devices are connected, the glucose meter will display "PC", indicating communication mode. The meter's stored readings will be transferred to HealthPAL.
3. After a few seconds HealthPAL will receive the readings, then display and audibly announce the most current glucose reading. ("**Your glucose reading is XXX.**")

Measurement(s) will be stored in HealthPAL's memory and transmitted to server automatically when HealthPAL is inactive for 3 minutes .

Always follow manufacturer instructions for operation and safety for any accessory health monitors used with HealthPAL.

"Your Blood Glucose Reading is: One-Hundred-Seven."

